Instructions for use

Operating temperatures : between 10°C and 40°C
Storage temperatures : between -20°C and 70°C
Keep out of the reach of children and animals.

Do not swallow.

If swallowed, call 112.

Warning: fragile (contains a glass tube).

Avoid contact with clothes.

Do not point at another person while deflating the bag.

In case of contact of the reactant with skin or eyes, rinse thoroughly with water.

This is a single-use breathalyser; it must be used only once, regardless of the results.

Dispose of the product once used.

Because the test contains chemical products in the reactive tube, you must dispose of the product with normal household waste. Check the expiry date is valid before use.

The results of the disposable breathalyser cannot be used to legally dispute those resulting from an official breath test. Decisions made by the driver, on the basis of the results of this test, remain his or her sole responsibility. With a blood alcohol content of 0,5 g/l of blood, the risk of a car accident is multiplied by three. The producer and distributors of this product cannot be held responsible for inappropriate use of the product and/or for failure to exercise the necessary precaution and correctly follow instructions.

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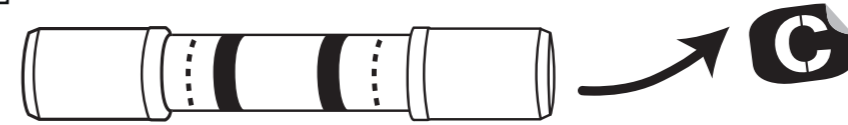
Instructions for Single-Use ALCOHOL BREATHALYSER

Wait at least 10 minutes after drinking alcohol or other substances (mouthwash, medicine...) before you take the breath test.

The maximum blood alcohol level is reached about 45 min to 60 min after last consuming alcohol.

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Insert the tube into the mouth of the bag.

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Press all the air out of the bag over a 15 second period, then remove the tube.

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- Wrap your hand around the tube. Depending on the colour of the round label on the right, keep the tube in your hand for the following amount of time:
White: 1 min. Red: 3 mins.

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Reading the result : Open your hand and immediately check the colour of the crystals !

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- RED colour = breath alcohol level greater or equal to 0,25 mg/litre of air equivalent to 0,5 g/litre of blood.
YOUR ALCOHOL LEVEL IS ABOVE THE LEGAL LIMIT: DO NOT DRIVE.

- No RED colour = breath alcohol level less than 0,25 mg/litre of air equivalent to 0,5 g/litre of blood.

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